BLOOD-BANK C	2101512
BLOOD-BANK D	2101514
BLOOD-BANK E	2101511
BLOOD-BANK F	2101513

**BANCOS DE SANGRE
BLOOD BANK STORAGE CABINETS**

INDICE

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GENERAL INFORMATION

- 1) Handle the parcel with care. Unpack and check that the contents coincide with the packing-list. If any part is damaged or missing, please notify to your J.P. Selecta supplier immediately.
- 2) Do not install or use the equipment without reading this handbook before.
- 3) This handbook must always be attached to the equipment and it must be available for all users.
- 4) If you have any doubts or enquiries, please contact with your supplier or J.P. Selecta's technical service.
- 5) IMPORTANT! J.P. SELECTA WILL NOT ACCEPT ANY EQUIPMENT TO BE REPAIRED IF IT IS NOT DULY CLEANED.**
- 6) If any modification, elimination or lack of maintenance of any device of the equipment by the user transgress the directive 89/655/CEE, the manufacturer is not responsible for the damage that can occur.
- 7) Do not use the equipment with liquids which can give off vapours capable of making explosive mixtures.

PACKING LIST

The standard equipment consists of the following components:

Piece	Part Number	Cant.
Blood-Bank C/E Shelves	2101512 / 2101511 --	1 2
Blood-Bank D/F Shelves	2101514 / 2101513 --	1 8
Set of 100 graphs	0131670	1
Instructions manual	80382	1

TECHNICAL FEATURES

Voltage supply 115-230V 50/60 Hz according to the characteristics plate indications.

Ambient temperature must not be below 22°C in Blood-Bank E and F equipments (2101511 & 2101513).

Part number	Litres capac.	Bags capac.	Internal dimensions cm			External dimensions cm			Consum (W)	Power (HP)	N° shelves	Weight (Kg)
			Height	Length	Width	Height	Length	Width				
2101512	600	200	136	58	69	207	70	82	350	3/8	5	132
2101514	1300	480	136	128	69	207	140	82	1100	3/8	10	190
2101511	600	200	136	58	69	207	70	82	350	3/8	5	132
2101513	1300	480	136	128	69	207	140	82	1100	3/8	10	190

EQUIPMENT DESCRIPTION

BLOOD-BANK C/D storage cabinets are designed to work at fixed temperatures at +4°C (Stability $\pm 1,5^\circ\text{C}$) and **BLOOD-BANK E/F** platelet incubators are designed for +22°C fixed temperatures. Equipped with an hermetic compressor mounted on shock absorbers with evaporator and forced air condenser, air cooling system which stops on opening the door, temperature sensors housed in similar conditions to the blood bag, inner LED light, double insulated anti-mist glass door and a circular chart recorder.

Stainless steel AISI 304 external cabinet. Inner lining also in stainless steel. Double glazed antimist, reversible lockable door, easy to change, self close door it left ajar.

Fan air circulation in order to reach homogeneous temperature distribution. External alarm ability. Circular graphic register. Pt100 probe. Safety thermostat. Temperature controlled by microprocessor which thought display allows view and control following parameters:

- Security password
- Set temperature
- Set alarms
- Calibration
- Start up cooling motor delay.

INSTALLATION

Avoid installing the equipment near heat sources (sunny places, heating, etc). Leave a space of about 10 cm between the wall and the back of the cabinet. Place the equipment on a hard horizontal leveled floor; if it is necessary use the adjustable foot. Do not start up before 4 hours.

VERY IMPORTANT BEFORE THE FIRST STARTING UP

Connect the equipment at mains.

The default temperature set is 4°C (22°C in the case of models E/F). Do not fill the equipment if it has not reached this temperature.

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
Be sure that the voltage supply is the same as the one indicated on the characteristics plate.

Do not use the equipment if it is not earthed.

If you have to change the plug cap, bear in mind the following:

Blue cable:	Neutral
Brown cable:	Phase
Yellow/green cable:	Earth

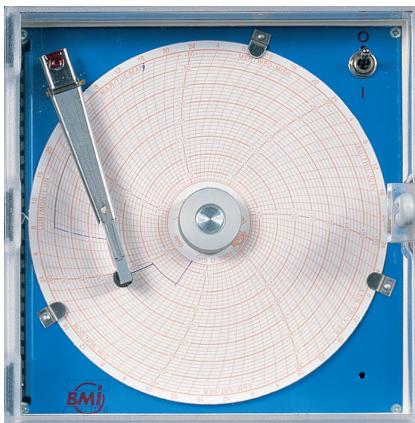STARTING UP THE RECORDER

1. Open the recorder lid by using the key
2. Remove the needle
3. Unscrew the shaft handle and remove the paper
4. Pull out the shaft to remove the recorder
5. Insert the battery into the rear part
6. Insert the recorder again
7. Put the needle again
8. Close again the recorder lid

Use a good quality alkaline battery to ensure a duration of more than 6 months.

MAINTENANCE

Before removing the casing, disconnect the equipment from the mains.

The manipulation of the internal electronic circuits of the equipment by unqualified personnel can cause irreparable damage.

Be sure that the equipment is repaired by a J.P.SELECTA authorized technical service.

BASIC MAINTENANCE

- Change recorder battery every 6 months (see "installation")
- Change recorder paper every week (see "installation")
- Change pen from recorder every 5 years (90 meters)
- Clean the evaporator of the cooling group at least once a year.

CLEANING

Use alcohol with a non abrasive cloth for the cleaning of the different pieces of stainless steel and plastics.

OPERATION


CONTROL PANEL:

FEATURES:

- Large display
- Integrated keys
- Camera probe and evaporator probe (PTC / NTC)
- Micro port entry
- Alarm buzzer
- TTB MODBUS port for connection to peripherals

OPERATION INDICATIONS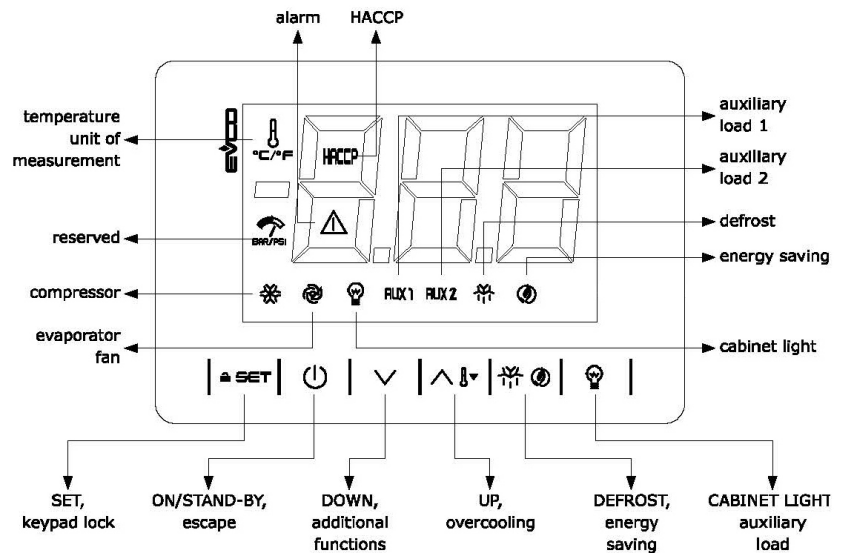

4.1 Switching the device on and off

1. If POF = 1 (default), touch the ON/STAND-BY key for 2s.

If the device is switched on, the display will show the P5 value ("cabinet temperature" default); if the display shows an alarm code, see the section *ALARMS*.

LED	ON	OFF	FLASHING
	compressor on	compressor off	- compressor protection active - setpoint being set
	evaporator fan on	evaporator fan off	evaporator fan stop active
	cabinet light on	cabinet light off	cabinet light on by digital input
	defrost or pre-drip active	-	- defrost delay active - dripping active
	- energy saving active - low consumption active	-	-
	view temperature	-	overcooling or overheating active
	alarm active	-	-

ERROR CONDITIONS

COD.	DESCRIPTION	RESET	TO CORRECT
Pr1	cabinet probe alarm	automatic	- check P0
rtc	clock alarm	manual	set date, time and day of the week
AL	low temperature alarm	automatic	check A0, A1 and A2
AH	high temperature alarm	automatic	check A4 and A5
id	open door alarm	automatic	check i0 and i1
PF	power failure alarm	manual	- touch a key - check electrical connection
C1t	compressor thermal switch alarm	automatic	check i5 and i6

DESVIATION OF TEMPERATURE

The default working temperature is 4°C ±2°C in Blood Bank C/D and 22°C ±2°C in Blood Bank E/F. If the temperature rises outside the working range, either above or below, an error occurs which is displayed on the screen (See previous table).

Both the working temperature and the upper and lower limits are programmable as explained in the section "Modification of parameters".

NOTES ON ALARMS

- The acoustic signals that cause the error conditions can be stopped by pressing the ON/STAND-BY key.
- There is the possibility of using one of the board relays to activate an external alarm in the event that the error condition occurs (See connection diagrams).

GUARANTEE

This product is guaranteed for one year. The guarantee does not cover damages caused by an incorrect use or causes beyond the control of J.P. SELECTA, s.a.u.

Any manipulation of the equipment by unauthorized personnel by J.P. SELECTA, s.a.u. cancels the guarantee automatically.

Notice to customers:

The product is made up of various components and various materials that must be recycled or, failing that, deposited in the corresponding debris removal sites when the product's life has been completed or when otherwise it is necessary to dispose of it. To do this, the end user who acquires the product must know the current regulations of each municipality and / or locality based on the waste electrical and electronic equipment. The user who acquires this product must be aware of and responsible for the potential effects of the components on the environment and human health as a result of the presence of hazardous substances. Never place the product in a conventional container of citizen scope if a previous dismantling and knowledge of the components that incorporates. If you do not know the procedure to follow, consult with the city council for more information.