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This product is guaranteed for one year from the date of acquisition. The guarantee does not cover damages caused by a misuse of the equipment.

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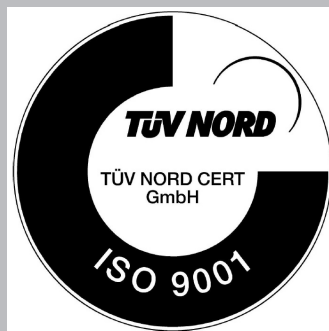
*Read this manual before starting up the equipment.
Do not switch on the machine if it is not earthed.
Do not use the equipment to seal two plastics.*

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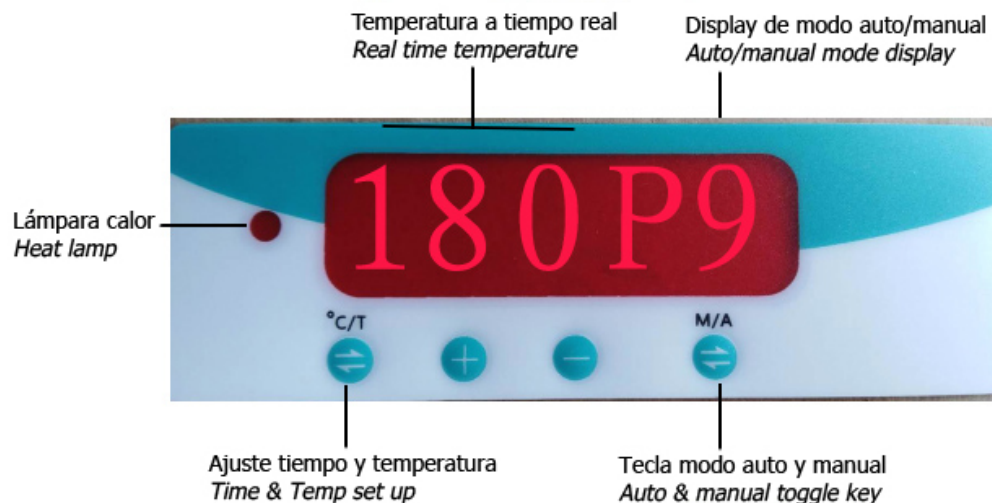
INSTRUCTIONS

1. Turn the power switch on, the left screen will show the real time temperature and the right screen will show the paper feeding status. **A** means automatic and **P** means continue paper feeding.
2. The machine will stay thermostatic when it rises to the present temperature. Generally, it comes at **180°C** from factory. **DO NOT CHANGE THE SETTINGS UNLESS NECESSARY.**
3. Press the **M/A** key to change the paper feeding status. When the screen show **A9**, no paper in within 9s and the machine will enter the automatic standby mode.
4. Press the **M/A** key again, the screen will show **P9**. The machine will continue working.
5. Press the **°C/T** key. You can increase or reduce temperature by using the + or - keys.
6. Press the **°C/T** key for 3sec. The screen will show **00**, and you can adjust the temperature deviation by pressing + or -.
7. Press the **°C/T** key for 3sec. Then press the **M/A** key. The screen will show **PO**. You can adjust the standby time of paper feeding by pressing + or -.
8. Press the **M/A** key again and you will go back to the original status and show the real time temperature.

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FEATURES

- Stainless steel or painted shell
- Automatic and continuous mode
- Heater
- Light control system
- Temperature control system
- Driver structure